


BioTherm 5 Dry Ice Packing Instruction

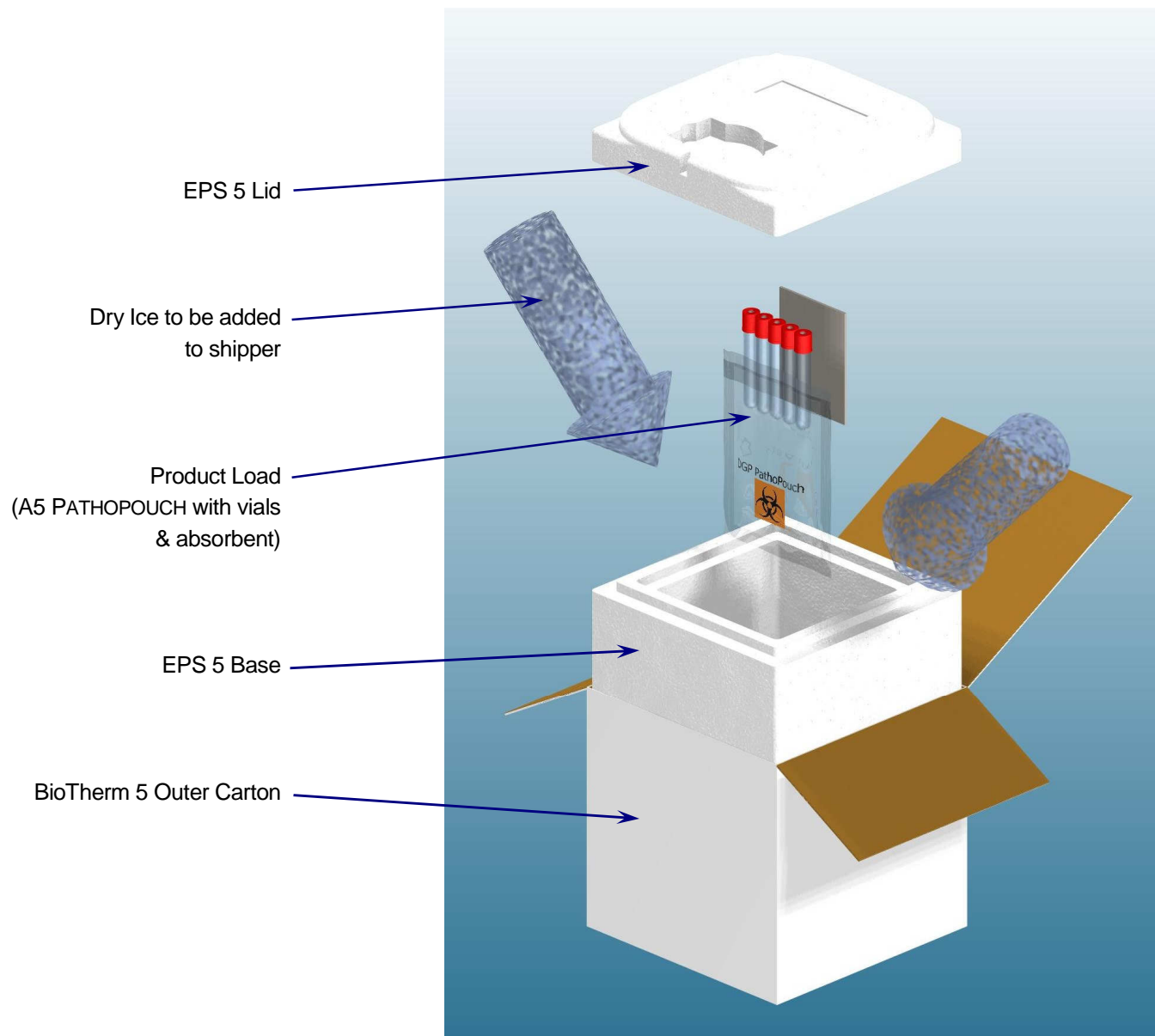

Caution

Please observe all appropriate local safety precautions when handling and dispensing Dry Ice (Solid CO₂)

Assembly Instructions

Place the product inside the PATHOPOUCH and seal by removing the foil strip and pressing the opening of the pouch together firmly.

Place the PATHOPOUCH in the centre of the EPS 5 base. Fill the EPS 5 base to the top with Dry Ice (Solid CO₂) pellets.

Apply the EPS 5 lid, then close and seal the Outer Carton. The system is now ready to be shipped.